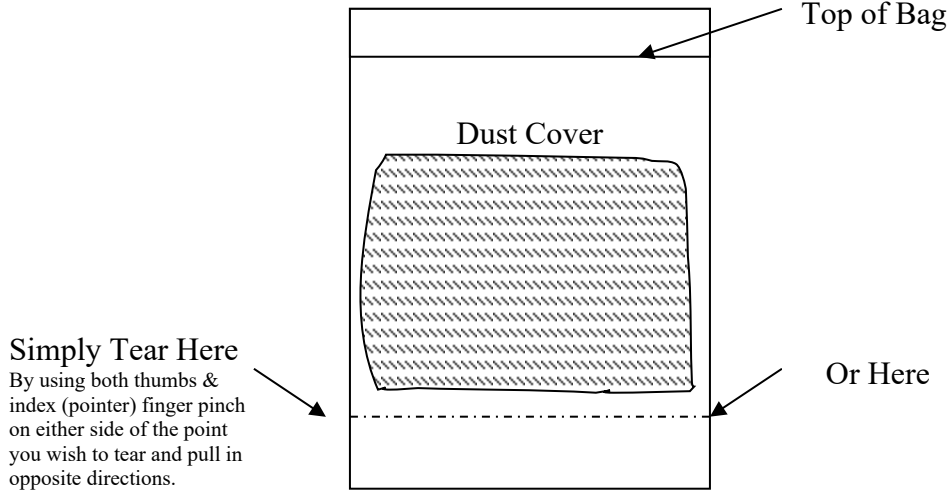


<b>Brand Name of Product</b>	Dust Covers
<b>Generic Name of Product</b>	Dust Covers
<b>Product Code Number(s)</b>	<ul style="list-style-type: none"> <li>• Heat Seal               <ul style="list-style-type: none"> <li>○ SM0610, SM0812, SM1015, SM1220, SM1622, SM1630, SM2430</li> </ul> </li> <li>• Self-Seal               <ul style="list-style-type: none"> <li>○ SE0610, SE812, SE1015, SE1220, SE1622, SE1630, SE2430</li> </ul> </li> </ul>
<b>Purpose of Product</b>	Protective plastic bag used to protect items from environmental factors such as moisture, dust, and lint.
<b>Range of Applications for Product</b>	Variety of items that require protection in storage.
<b>Key specifications of product</b>	Variety of sizes available.

<b>Shipping &amp; Storage</b>	
<b>Shipping Conditions &amp; Requirements</b>	
<b>Storage Conditions</b>	
<b>Packaging Contents</b>	
<b>Shelf Life</b>	

<b>Instructions for Using Product</b>	
<b>Description of Use(s)</b>	Protective plastic bag used to protect items from environmental factors such as moisture, dust, and lint.
<b>Preparation for Use</b>	
<b>Diagrams (drawings, pictures)</b>	 <p> <b>Simply Tear Here</b>        By using both thumbs &amp; index (pointer) finger pinch on either side of the point you wish to tear and pull in opposite directions.     </p>
<b>Steps for Use of Product</b>	<ul style="list-style-type: none"> <li>• Dust cover is sealed using either a heat sealer designed to seal plastic to plastic or an alternative method that is similarly effective. A self-sealing cover also may be used (it will take around 3 minutes for the seal to set up).           <ul style="list-style-type: none"> <li>○ For optimum heat seal of your easy-tear material, a temperature range between 107°C (225°F) and 110°C (230°F) works best.</li> </ul> </li> <li>• The lot or load control number and expiration statement should be visible through the dust cover of the item that is placed with, or an additional label should be used on the dust cover, placed on the outside.</li> <li>• Allow items to cool down to room temperature prior to placing into dust cover.</li> <li>• Dust covered items should never be carried under the arms.</li> <li>• Dust covered items should never be transported/carried by grabbing/holding the end of the bag during transportation (this action can comprise the seal and cause undue stress and tearing).</li> <li>• Proper care and handling of dust covered items should be observed at all times. Carrying item in both hands is the recommended practice.</li> </ul>

	<ul style="list-style-type: none"> <li>• In order to open, using both thumbs &amp; index (pointer) finger pinch on either side of the point you wish to tear and pull in opposite directions and pull apart.</li> <li>• Once the seal on the edge is broken, the bag will easily tear all the way across.</li> </ul>
<b>Interpretation of Results</b>	
<b>Contraindications of Test Results</b>	
<b>Documentation</b>	
<b>Special Warnings and Cautions</b>	<ul style="list-style-type: none"> <li>• Placing a dust cover on a package that is not cool and dry could result in condensation inside the dust cover and compromise the package contents.</li> <li>• Dust covers are not a sterile barrier and do not extend the stated shelf-life of the contents.</li> </ul>
<b>Disposal</b>	Single-use. Dispose of according to facility's policy.

<b>Reprocessing Instructions</b>	
<b>Point of Use</b>	
<b>Preparation for Decontamination</b>	
<b>Disassembly Instructions</b>	
<b>Cleaning – Manual</b>	
<b>Cleaning – Automated</b>	
<b>Disinfection</b>	
<b>Drying</b>	
<b>Maintenance, Inspection, and Testing</b>	
<b>Reassembly Instructions</b>	
<b>Packaging</b>	
<b>Sterilization</b>	
<b>Storage</b>	
<b>Additional Information</b>	

<b>Related Healthmark Products</b>	
<b>Other Product Support Documents</b>	
<b>Reference Documents</b>	Sterilization Packaging Brochure, Sterilization Packing Price List
<b>Customer Service Contact</b>	Healthmark Industries Company, Inc. 18600 Malyn Blvd. Fraser, MI 48026 1-586-774-7600 <a href="mailto:healthmark@hmark.com">healthmark@hmark.com</a> hmark.com